

# Bemis® 800cc Hydrophobic Suction Canister

Approved for the **ATMOS S 041** NPWT pump



**Intended Use:** For the collection and disposal of aspirated fluids.


## Directions for Use:

1. Place lid on canister and press firmly around entire perimeter.
2. Apply pour spout cap firmly over pour spout.
3. Firmly attach canister connection hose (included) to vacuum port and attach ATMOS wound drain to patient port.
4. Check all connections for proper seal. Test the assembly for vacuum leaks by turning on ATMOS S 041 and occluding the ATMOS wound drain tubing with finger or thumb.
5. To prevent fluid contamination to ATMOS S 041, make sure canister connection hose is attached to vacuum port.

**Notice:** These directions are neither intended to or disclose directions for other ATMOS products. For further directions for ATMOS Wound Drains and ATMOS Wound Dressing Kits please reference the directions included in their packaging.

## Disposal:

1. Turn off ATMOS S 041 and disconnect all tubing.
2. Seal vacuum and patient ports with attached port caps.
3. Remove canister from bracket and transport to disposal area. **Do not lift canister by lid.** The weight of the contents may cause the lid to separate from the canister.
4. Dispose of according to hospital policy.

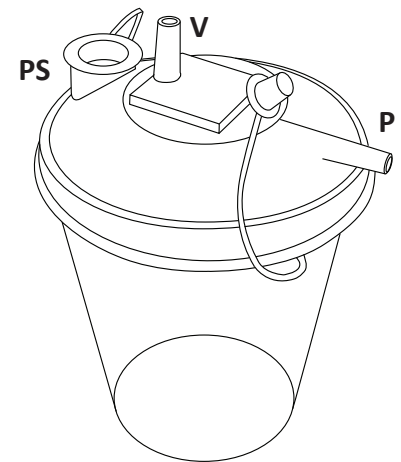

**P - Patient Port**  
**V - Vacuum Port**  
**PS - Pour Spout**

## ⚠ CAUTION

- **Single use only. Do not attempt to clean, sterilize or reuse canister.**
- Do not exceed vacuum level of 25" (640mm) Hg.
- Not recommended for use in liposuction procedures.
- Do not apply continuous vacuum longer than 24 hours.
- Not intended as a measuring device, only for general reference, not specific measurement.
- Canister contents are considered potentially hazardous, use appropriate PPE and handle accordingly.
- Check expiration date on canister for use-by date.

**Loss of Vacuum:** Check that vacuum is on, canister is properly sealed and that all connections are tight and tubing is not kinked. If loss continues, replace canister. Report any product malfunctions or complaints to ATMOS Inc. using the contact information below.

**Notice:** Only use ATMOS approved suction canisters. Use of unapproved suction canisters may void warranty.

**To reorder call us Toll Free at 1-866-862-8667.**  
**Use ATMOS part number 314.0020.0.**

Portions of this document are used with permission from Bemis Manufacturing Company. Bemis is a registered trademark of Bemis Manufacturing Company. Bemis Manufacturing Company does not support or endorse ATMOS Inc./ATMOS GmbH. or associated products. ATMOS and the ATMOS logo are trademarks of ATMOS Inc./ATMOS GmbH.

**ATMOS Inc.**

3717 Huckleberry Road  
Allentown, PA 18104

Toll free: 1-866-862-8667  
Fax: 1-610-351-6827

info@atmosmed.com  
www.atmosmed.us

